

✓ **REFERENCE**

Calibrators KIT	BUREK-000	5 x 1 ml	2-8°C
Beta 2 Microglobulin in synthetic biological fluid standardized from the international standard ERM-DA470/IFCC <sup>1</sup> , sodium azide (< 1g/l)			
Lot #	20J02		
Expiry date	04/2021		
Control date	02/10/2020		
Quality control report #	DGM-QAC-REP-20150		
Document prepared and signed by	L. Ginneberge		

✓ **SAMPLES AND REFERENCE VALUES**

See the corresponding reagents technical sheet.

✓ **COMPOSITION**

Beta 2 Microglobulin calibrators are synthetic biological fluids containing human Beta 2 Microglobulin at fixed value diluted in buffer containing stabilizers and sodium azide (<1g/l) as preservative.

✓ **PRINCIPLE OF TEST**

The latex particles in colloidal form are stabilized with anti-β2M antibodies directed specifically against β2M. The reaction of these particles with β2M, present in a biological sample, causes the specific agglutination of the latex particles. This agglutination is directly proportional to the β2M concentration of the sample.

✓ **PRECAUTIONS**

For in vitro single diagnostic use. To be handled by entitled Personnel.  
 Products from human source were tested and found free from HBsAg and antibodies to HCV and HIV but this material should be treated just as carefully as potentially infective.  
 Products containing sodium azide have to be handled with care; avoid ingestion and contact with skin and mucous membranes. Sodium azide may react with lead or copper plumbing to form highly explosive metal azides.

✓ **ANALYTICAL PERFORMANCES**

See the corresponding reagents technical sheet.

✓ **PREPARATION AND REAGENTS STABILITY**

The calibrators are ready for use; once opened, they are stable until expiry date if stored stoppered in appropriate temperature conditions and without any contamination.

✓ **METHOD OF ANALYSIS AND CALCULATION**

See the corresponding reagents technical sheet.

✓ **QUALITY CONTROL**

**Accuracy and reproducibility:** Analytical performances can be checked with the internal quality control serum of the laboratory.

**Calibration:** Calibration curve and stability of calibration curve can be validated with the DiAgam calibration controls (B2COS-003, B2CON-003, A1COS-002, A1CON-002).

Calibrate when the quality control results are outside acceptable range (contact the manufacturer if the deviations subsist), when the reagent lot number changes or when government regulations require.

✓ **BIBLIOGRAPHY**

Certification of proteins in the human serum. Certified Referenced Material ERM®-DA470k/IFCC. I. Zegers et al. <http://irmm.jrc.ec.europa.eu/>



BUREKFTEN 02/10/2020 v01

Beta 2 Microglobulin Urinary	CAL 1	CAL 2	CAL 3	CAL 4	CAL 5
	mg/l	mg/l	mg/l	mg/l	mg/l
	<b>0.250</b>	<b>0.500</b>	<b>1.5</b>	<b>5.0</b>	<b>8.0</b>

Values assigned from the reference ERM-DA470k/IFCC.